Face Mask Safety Precautions

COVID-19 BASICS *for* SCHOOL NUTRITION PROGRAMS

A Collaborative Resource from
Center for Ecoliteracy, LunchAssist
and Eat REAL

Putting on a Mask is as Easy as... 1, 2, 3!


Step 1:

Clean hands for 20 seconds with soap and water or hand sanitizer.

Step 2:

Inspect the mask and check that it is clean and there are no tears or holes. Determine which side of the mask is the top, and which side is the front.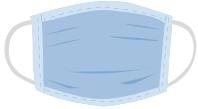

Step 3:

Masks with ear loops: Hold by the ear loops and place around each ear.

Masks with ties: Hold by the ties. Bring the mask to nose level. Secure ties with a bow behind your head.

Masks with bands: Pull the straps over your head and secure comfortably.

Do not touch mask while wearing it, even when talking to others. If you do, wash or sanitize hands and change gloves.

Tip: Make sure masks are fitted over your face and nose with no gaps between your face and mask.

REMOVING YOUR MASK

Clean hands before removing and do not touch the front of your mask.

Masks with ear loops: Hold both ear loops and gently lift and remove mask.

Masks with ties: Untie bottom bow first then untie top bow and pull mask away from you.

Masks with bands: Remove bottom band first then the top band and pull mask away from you.

CARING FOR YOUR MASK

Fabric masks: Clean with soap and hot water after each use.

Single-use or fabric masks that can't be immediately washed: Store in a paper bag for 3-4 days before reusing.

Replace single-use masks if they become dirty, damp, wet or hard to breathe through.

This is unofficial guidance based on best practices as of 4/3/20 for using face masks in school nutrition settings.

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